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User Leaflet

## <sup>13</sup>C UREA Tablets

**Description:** UREA Tablet, enriched with <sup>13</sup>C, 75 mg.

**Indication:** For the diagnosis of *Helicobacter pylori* (*H. pylori*) presence in the human stomach by breath test, as an aid in the initial diagnosis and post treatment monitoring of *H. pylori* infection.

For use with a suitable kit (according to the instructions for use in the leaflet provided with the kit).

**Content:** Each tablet contains 75 mg of <sup>13</sup>C-Urea

Inactive ingredients:  
Sodium Benzoate, NF, Povidone, USP (Kollidon 90F), Microcrystalline Cellulose, NF (Avicel 102), Colloidal Silicon Dioxide, NF (Aerosil 200).

#### Instructions for Use:

##### Preparation of the drink:

Dissolve the <sup>13</sup>C-Urea tablet in 150 to 200 mL of tap water together with citric acid ("Citrica") supplied in a separate sealed pouch in a single drinking cup.  
Stir thoroughly with the provided straw for 2 to 3 minutes until the Citrica powder and the urea tablet are completely dissolved. Small particles might still be seen in the solution after stirring.

Alternatively, the tablet may be dissolved in 150 to 200 mL of tap water with 1.4 g of citric acid or in a glass of orange juice without Citric acid.

Drink the content of the drinking solution with a straw.

Drink the drinking solution within two minutes.

##### Note:

If substantial particulate matter is still present after 5 minutes of stirring, discard the solution and repeat the procedure with a new tablet.

Administer the test drink within two hours of preparation, as this is the maximal time to maintain solution stability.

##### Recommendations before the use:

The patient should have fasted at least 1 hour before administering the solution.

The patient should not have taken proton pump inhibitors and antimicrobials, within two weeks prior to administering the test.

##### Recommendations at the time of use:

The breath test should be performed within 30 minutes after drinking the solution.

During these 30 minutes no eating, drinking, or smoking is allowed.

##### Warnings:

For diagnostic use only.

The dissolved solution is taken orally as part of the diagnostic procedure.

In the case of accidental overdose – immediately contact the poison control emergency center.

A negative result does not rule out the possibility of *H. pylori* infection. False negative results can occur with this procedure.

A false positive test may occur due to presence of other microorganisms like *Helicobacter heilmanni*.

A false positive test could occur in patients who have achlorhydria.

If clinical signs suggest *H. pylori* infection, retest with a new sample or an alternate method.

##### Side effects:

In the event that you experience side effects not mentioned in this leaflet, or if there is a change in your general health, consult with your doctor immediately.

**Storage:** At room temperature, below 25°C.

##### Manufactured by:

Taro Pharmaceutical Industries Ltd.  
14 Hakitor St., P.O.B. 10347, Haifa Bay

##### For:

Exalenz Bioscience Ltd.  
4 Hamaayan St. Modiin 71700

The format of this leaflet was determined by the Ministry of Health and its content was checked and approved by it in March 2010.

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